

Smile Brightening Care Kit

Opalescence Go® 10-PACK Professional Whitening



- Working from the inside out, the active ingredients penetrate your teeth to break down the discolored molecules, removing stains and whitening your smile
- Contains potassium nitrate and fluoride (shown to strengthen enamel, decrease sensitivity, and help prevent cavities)

Box Includes: 10 application tray sets (upper & lower teeth) for 10 day whitening treatment

[More details at opalescence.com]

Opalescence® Whitening Toothpaste



- Available in Opalescence Original formula with fluoride
- Safe for multiple uses per day
- Comes in Cool Mint flavor

[More details at opalescence.com]

CALL: 888-60-DENTAL
westerndental.com

**Western Dental®
& Orthodontics**

Smile Brightening Care Kit Instructions



***For 10% Hydrogen Peroxide Kit Instructions**
(For 15% Hydrogen Peroxide Kit, decrease wear time to 15 to 20 minutes)

[Reference: Opalescence Instruction Guide]



1. Remove from package



2. Center tray on arch



3. Gently suck down or swallow



4. Remove outer tray



5. Suck down or swallow again



6. Wear for 30 to 60 minutes*

Opalescence® Whitening Toothpaste



- **Adults and children over 6 years of age:** Brush as directed by a dentist or physician
- **Children of 6 years and younger:** Use a pea-sized amount for supervised brushing to minimize swallowing until capable of using without supervision
- In case of intake of fluoride from other sources, or for children under 2 years of age, consult a dentist or doctor

[Reference: Opalescence Instruction Guide]

Instructions have been provided for patient convenience. Please refer to the manufacturer's product guides included in their package for details.

